GEBRAUCHSANWEISUNG / INSTRUCTIONS FOR USE / MANUEL D'INSTRUCTIONS / ISTRUZIONI PER L'USO /  
INSTRUÇÕES DE UTILIZAÇÃO / INSTRUCCIONES DE USO / 使用说明书 / उपयोग के निर्देश / 取扱説明書 /  
ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ  
Instrumente / Instruments / Instruments / Strumenti / Instrumentos / Instrumentos / 医疗器械 / प्रपत्र / 器材 /  
Инструменты

REF X0008 LOT 2021-02-09

DE

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EN

**1. INTENDED USE**  
All instruments provided by NGMedical are reusable surgical instruments with the purpose of enabling the implantation of an NGMedical implant. The surgical technique of each implant describes the instruments to be used. The instruments may only be used in direct connection with implants from NGMedical and as described in the respective surgical techniques.

**2. SCOPE**  
These instructions for use apply to all NGMedical instruments.

**3. IMPLANTS**  
All surgical techniques are available from NGMedical on request. The article numbers of the implants can be found in the corresponding surgical techniques.

**4. INDICATIONS/CONTRAINDICATIONS**  
The NGMedical instruments are only used in direct connection with the associated implant. Therefore, the indications/contraindications of the respective implant apply. The indications/contraindications of the associated implant can be viewed in the respective instructions for use.  
All instructions for use are available from NGMedical on request.

**5. POSSIBLE COMPLICATIONS**  
Possible complications are:

- Loss of function of the instrument
- Laser marking becomes illegible
- Corrosion
- Damaged surfaces
- Split-offs
- Contamination
- Fissures and sharp edges
- Deforming

Sort out the instruments indicating one or several complications. All instruments must be checked for their functionality according to the surgical technique before the start of an operation. Please also note the document "Reprocessing Instructions Instruments" (REF X0200).

**6. REPROCESSING**  
All NGMedical instruments are placed on the market non-sterile and must be subjected to reprocessing and sterilization before each use in accordance with the document "Reprocessing Instructions Instruments" (REF X0200). The Reprocessing Instructions Instruments are available from NGMedical on request.

**7. FURTHER INFORMATION**  
For further information, please contact:

NGMedical GmbH  
Talsperrenblick 5  
66620 Nonnweiler  
GERMANY  
mail@ngmedical.de  
www.ngmedical.de  
+49 (0) 6873 99997-0

	Consult Instructions for Use		Non-Sterile
	Keep Dry		Manufacturer
	Reference-No.		Serial Number
	UDI (Unique Device Identification)		Medical Device
	Do not use if package is damaged		Date of manufacture

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